MANUALE D'USO
DEAMBULATORE ARTICOLATO
COD. 01.06.100

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USER MANUAL
ONE BUTTON FOLDING WALKER
CODE 01.06.100

INTRODUCTION

Thank you for purchasing one of our products. The walkers have been designed and manufactured to meet all your needs for practical, correct and safe use. This manual contains some suggestions for a correct use of the device and valuable advice for your safety. We recommend to read all the manual carefully before using the item. In case of any doubts, please contact the dealer, who will be able to advise you correctly.

Note: Check all parts to verify it won't be any shipping damage. In case of any damages, do not use it. Contact the dealer for further instructions.

DECLARATION OF CONFORMITY

DEMARTA VIRGINIO SAS declare under sole responsibility that the products that belong to walker range are in accordance with the European Directive on Medical devices (93/42/EEC).

DEMARTA VIRGINIO SAS guarantees and declared under sole responsibility what follow:

- The devices under discussion satisfy the essential requisites requested by the I enclosure directive 93/42/CEE as laid down by the VII enclosure of the above mentioned Directive.
- The devices under discussion belong to class I.
- It is NOT PERMITTED to use this devices outside the intended use envisaged by DEMARTA VIRGINIO SAS.
- The articles also comply with all European standards applicable to them.

USE

The walkers are indicated in the walk rehabilitation for elderly or subjects with functional limitations of various kinds. They offer stability, ease of use and improve the sense of balance.

This aid is designed for a single user, in indoor environments.

WARNINGS

- The walker should only be used by patients with a suitable motor capacity which ensures their use without risks.
- Advise with physician or therapist to determinate the correct adjustment and the correct use.
- Physician or therapist should contribute to height adjustment for the maximum support, for a correct brake activation and for a general assembly.
- The walker can be used only on a hard, flat surface.
- Do not use the walker to climb the stairs.
- Pay particular attention to wet surfaces.
- All the tips must be in contact with the floor at any time during use to ensure the proper balancing of the walker and maximum safety.

DESCRIPTION

- Height-adjustable foldable aluminum frame with horizontal stabilizing bar.
- Four legs equipped with anti-slip tips.
- Folding can easily be done using the locking pin located at the center of the upper bar.
- Ergonomic and padded knobs.
- Maximum Capacity: 120 Kg.

TECHNICAL FEATURES

Code	Length	Width	Height	Weight Capacity	Net Weight	Carton Size
Measures in Cm						
01.06.100	46	56.5	75/93	120	2.2	58x15x80

ASSEMBLY

- To open the frame, press the central locking pin until the side supports are completely open.
- Adjust the height of the walker before use. Press the button on the leg of the walker and pull up or down the lower leg to find the hole that corresponds to the most appropriate height. Do the same operation on all 4 legs so they are at the same height.
- Check that the rubber tips are correctly fixed. In the event that the tips are worn, they must be replaced immediately.

USE

- Start pushing when you are in an upright position with both hands on the handle.
- Make sure your hands are dry before using the rollator to avoid the risk of slipping.
- Walking maintaining an upright position with his eyes facing forward and not downward.
- Do not push the walker too far away from the body, this may cause falls.
- Do not use the walker to overcome steps.

CLEANING

Use a damp cloth with mild soap, the water temperature not above 30°C. Dry well before use. If necessary, disinfect the product using a common disinfectant detergent. Do not use water and/or steam jet washing machines. Must not use acid, alkaline or solvent products such as acetone.

GENERAL DISPOSAL CONDITIONS

When disposing of the device, never use housing solid waste collection systems.

It is necessary to dispose of the device through the common ecological islands common for the recycling of used materials.

WARRANTY

DEMARTA products are guaranteed from material or manufacturing faults for 2 years from the purchasing date.

DEMARTA does not guarantee for damages caused by the following condition: natural disaster, not authorized maintenance and failure to adhere to the guidelines and instructions for use.