MANUALE D'USO  
ROLLATOR BELLATOR  
COD. 01.06.410

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USER MANUAL  
ROLLATOR BELLATOR  
SKU 01.06.410

## INTRODUCTION

Thank you for purchasing one of our products. The rollator have been designed and manufactured to meet all your needs for practical, correct and safe use. This manual contains some suggestions for a correct use of the device and valuable advice for your safety. We recommend to read all the manual carefully before using the item. In case of any doubts, please contact the dealer, who will be able to advise you correctly.

**Note: Check all parts to verify it won't be any shipping damage. In case of any damages, do not use it. Contact the dealer for further instructions.**

## DECLARATION OF CONFORMITY

DEMARTA VIRGINIO SAS declare under sole responsibility that the products that belong to walking aids are in accordance with the European Directive on Medical devices (93/42/EEC). DEMARTA VIRGINIO SAS guarantees and declared under sole responsibility what follow:

- The devices under discussion satisfy the essential requisites requested by the I enclosure directive 93/42/CEE as laid down by the VII enclosure of the above mentioned Directive.
- The devices under discussion belong to class I.
- It is NOT PERMITTED to use this devices outside the intended use envisaged by DEMARTA VIRGINIO SAS.
- The articles also comply with all European standards applicable to them.

## INTENDED USE

The rollators are indicated in the walk rehabilitation for elderly or subjects with functional limitations of various kinds. They offer stability, ease of use and improve the sense of balance. This aid is designed for a single user, in indoor or outdoor environments.

## WARNINGS

- The rollator must be used only by patients with a suitable motor ability that guarantees their use without risks.
- Consult your doctor or therapist to determine the suitability of the product, the right adjustment and the correct use of the rollator.
- Your doctor or therapist should control the height-adjustment of the rollator, the correct activation of the brake, the correct assembly in general.
- Always make sure that the brake cable is properly secured.
- Always check the correct operation of the brake after each height adjustment.
- Failure to follow these simple warnings may result in personal injury.
- The rollator can only be used on a solid, flat surface.
- Do not use the rollator to climb stairs.
- Always lean on both hands to maintain balance.
- Pay particular attention to wet, icy or snowy surfaces.

- Do not use the rollator seat to transport people or things. The seat is intended for the user's seat with the brake on.
- All the wheels must be in contact with the floor at any time during use to guarantee the correct balance of the rollator.
- Check that both handle support tubes are properly inserted into the frame and secured with locking knobs.
- Do not hang anything on the rollator frame to avoid distortion of it.
- Before using the device, always check:
  1. That all parts of the rollator are securely locked.
  2. Check that all screws are tight.
  3. Make sure the seat is properly locked.
  4. Check for proper brake operation.

## DESCRIPTION

- Foldable aluminum frame with padded seat and backrest.
- Four wheels Ø 20 cm, two rotating front wheels and two fixed rear wheels with brake.
- It is equipped with a nylon storage basket.
- Height-adjustable padded knobs.
- Seat height from the ground 54 cm.

## TECHNICAL FEATURES

Code	Length	Inside Width	Width	Height	Seat Dimensions	Floor to Seat Height	Weight Capacity	Front Wheel	Rear Wheel	Net Weight	Carton Size
Measures in Cm											
01.06.410	72.1	45.7	64	82/92	45.7x25	48	120	20 (8")	20 (8")	7	75x27x85

## ASSEMBLY

### Handles assembly

Insert the handle tubes into the frame. Lock them by screwing on the knobs.

### Handles height-adjustment

Adjust the height of the handles using the side knobs.

Unscrew the knob, position the handles at the desired height. Turn the knob to fix it, tighten it well. Repeat the operation to the other side and make sure that both handles are positioned at the same height.

#### WARNING!

Make sure that the handles are mounted turned towards the rear wheels and that they are at the same height.

## Storage basket

The storage basket is fixed to the rollator frame.

### WARNING!

Use the storage basket to carry small items. Do not overload the basket to avoid imbalances and risks of overturning.

## Seat

Before sitting, place the rollator on a flat and solid surface and then **lock the brakes**. Make sure that the brakes is securely fastened and that the rollator is in a stable position.

### WARNING!

Distributing the weight over the entire seat. Do not sit pushing on one side only.

## USE

- Start pushing when you are in an upright position with both hands on the handle.
- Make sure your hands are dry before using the rollator to avoid the risk of slipping.
- Walking maintaining an upright position with eyes facing forward and not downward.
- Do not push the walker too far away from the body, this may cause falls.
- Do not use the rollator to overcome steps.

## CLEANING

Use a damp cloth with mild soap, the water temperature not above 30°C. Dry well before use. If necessary, disinfect the product using a common disinfectant detergent. Do not use water and/or steam jet washing machines. Must not use acid, alkaline or solvent products such as acetone.

## GENERAL DISPOSAL CONDITIONS

When disposing of the device, never use housing solid waste collection systems.

It is necessary to dispose of the device through the common ecological islands common for the recycling of used materials.

## WARRANTY

DEMARTA products are guaranteed from material or manufacturing faults for 2 years from the purchasing date.

DEMARTA does not guarantee for damages caused by the following condition: natural disaster, not authorized maintenance and failure to adhere to the guidelines and instructions for use.