MANUALE D'USO

DEAMBULATORE ASCELLARE - 01.06.500

OPTIONAL:

SELLINO IMBOTTITO - 01.06.500S

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SEAT SKU 01.06.500S

INTRODUCTION

Thank you for purchasing one of our products. The walker have been designed and manufactured to meet all your needs for practical, correct and safe use. This manual contains some suggestions for a correct use of the device and valuable advice for your safety. We recommend to read all the manual carefully before using the item. In case of any doubts, please contact the dealer, who will be able to advise you correctly.

Note: Check all parts to verify it won't be any shipping damage. In case of any damages, do not use it. Contact the dealer for further instructions.

DECLARATION OF CONFORMITY

DEMARTA VIRGINIO SAS declare under sole responsibility that the products that belong to walking aids are in accordance with the European Directive on Medical devices (93/42/EEC).

DEMARTA VIRGINIO SAS guarantees and declared under sole responsibility what follow:

- The devices under discussion satisfy the essential requisites requested by the I enclosure directive 93/42/CEE as laid down by the VII enclosure of the above mentioned Directive.
- The devices under discussion belong to class I.
- It is NOT PERMITTED to use this devices outside the intended use envisaged by DEMARTA VIRGINIO SAS.
- The articles also comply with all European standards applicable to them.

INTENDED USE

The walker are indicated in the walk rehabilitation for elderly or subjects with functional limitations of various kinds. They offer stability, ease of use and improve the sense of balance. This aid is designed for a single user, in indoor environments.

WARNINGS

- The walker must be used only by patients with a suitable motor ability that guarantees their use without risks.
- Consult your doctor or therapist to determine the suitability of the product, the right adjustment and the correct use of the walker.
- Your doctor or therapist should control the height-adjustment of the walker, the correct activation of the brake, the correct assembly in general.
- Failure to follow these simple warnings may result in personal injury.
- The walker can only be used on a solid, flat surface.
- Do not use the walker to climb stairs.
- Do not use the walker seat to transport people or things. The seat is intended for the user's seat with the brake on.
- All the wheels must be in contact with the floor at any time during use to guarantee the correct balance.

- Do not hang anything on the walker frame to avoid distortion of it.
- Before using the device, always check:
 1. That all parts of the walker are securely locked.
 2. Check that all screws are tight.
 3. Make sure the seat is properly placed.
 4. Check for proper brake operation.

DESCRIPTION

- Steel frame with two height-adjustable axillary supports.
- Four rotating wheels Ø 12.5 cm, rear wheels with brake.

Optional:

- it can be supplied with a padded seat.

TECHNICAL FEATURES

Code	Length	Inside Width	Width	Height	Seat Dimensions	Floor to Seat Height	Weight Capacity	Front Wheel	Rear Wheel	Net Weight	Carton Size
Measures in Cm											
01.06.500	68	45	51.5	115/150	52x21	59	120	12.5 (5")	12.5 (5")	10	73x28x91

ASSEMBLY

The frame of the walker is divided into 6 parts:

Base frame assembly:

Insert part nr. 3 (above, with the fixing knobs facing outwards) and part nr. 4 (below, with the fixing knobs facing outwards) in the side supports nr. 1 and nr. 2.

WARNING!

Tighten all four fixing knobs firmly.

Fix the auxiliary support:

Insert parts nr. 5-6 to the newly assembled frame.

WARNING!

Tightly screw the two side handles, this will also allow the height adjustment of the auxiliary supports.

Optional:

Padded seat:

Place and fix the seat to the horizontal tubes (parts 1 and 2), see image previous page.

Before sitting, place the walker on a flat, solid surface and lock the brakes.

WARNING!

Distributing the weight over the entire seat. Do not sit pushing on one side only.

USE

- Start pushing when in an upright position, with the auxiliary supports close to the body, and both hands on the handle.
- Make sure your hands are dry before using the rollator to avoid the risk of slipping.
- Walking maintaining an upright position with eyes facing forward and not downward.
- Do not push the walker too far away from the body, this may cause falls.
- Do not use the walker to overcome steps.

CLEANING

Use a damp cloth with mild soap, the water temperature not above 30°C. Dry well before use. If necessary, disinfect the product using a common disinfectant detergent. Do not use water and/or steam jet washing machines. Must not use acid, alkaline or solvent products such as acetone.

GENERAL DISPOSAL CONDITIONS

When disposing of the device, never use housing solid waste collection systems.

It is necessary to dispose of the device through the common ecological islands common for the recycling of used materials.

WARRANTY

DEMARTA products are guaranteed from material or manufacturing faults for 2 years from the purchasing date.

DEMARTA does not guarantee for damages caused by the following condition: natural disaster, not authorized maintenance and failure to adhere to the guidelines and instructions for use.