COD. 01.06.550S

COD. 01.06.550A

COD. 01.06.550T

## MANUALE D'USO DEAMBULATORI FISSI

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## USER MANUAL FIXED WALKERS

## INTRODUCTION

Thank you for purchasing one of our products. The walkers have been designed and manufactured to meet all your needs for practical, correct and safe use. This manual contains some suggestions for a correct use of the device and valuable advice for your safety. We recommend to read all the manual carefully before using the item. In case of any doubts, please contact the dealer, who will be able to advise you correctly.

**Note: Check all parts to verify it won't be any shipping damage. In case of any damages, do not use it. Contact the dealer for further instructions.**

## DECLARATION OF CONFORMITY

DEMARTA VIRGINIO SAS declare under sole responsibility that the products are in accordance with the European Directive on Medical devices (93/42/EEC).

DEMARTA VIRGINIO SAS guarantees and declared under sole responsibility what follow:

- The devices under discussion satisfy the essential requisites requested by the I enclosure directive 93/42/CEE as laid down by the VII enclosure of the above mentioned Directive.
- The devices under discussion belong to class I.
- It is NOT PERMITTED to use this devices outside the intended use envisaged by DEMARTA VIRGINIO SAS.
- The articles also comply with all European standards applicable to them.

## INTENDED USE

The walker are indicated in the walk rehabilitation for elderly or subjects with functional limitations of various kinds. They offer stability, ease of use and improve the sense of balance. This aid is designed for a single user, in indoor environments.

## WARNINGS

- The walker must be used only by patients with a suitable motor ability that guarantees their use without risks.
- Consult your doctor or therapist to determine the suitability of the product, the right adjustment and the correct use of the walker.
- Your doctor or therapist should control the height-adjustment of the walker, the correct activation of the brake, the correct assembly in general.
- Failure to follow these simple warnings may result in personal injury.
- The walker can only be used on a solid, flat surface.
- Do not use the walker to climb stairs.
- Do not use the walker seat to transport people or things. The seat is intended for the user's seat with the brake on.
- All the wheels must be in contact with the floor at any time during use to guarantee the correct balance.
- Do not hang anything on the walker frame to avoid distortion of it.

- Before using the device, always check:
  1. That all parts of the walker are securely locked.
  2. Check that all screws are tight.
  3. Make sure the seat is properly placed.
  4. Check for proper brake operation.

## DESCRIPTION

- Fixed steel structure.
- Four swivel wheels, two rear with stop brake.
- Axillary support (code 01.06.550S).
- Forearm support (code 01.06.550 A).
- Forearm support with table (code 01.06.550T).

## TECHNICAL FEATURES

Code	Length	Inside Width	Width	Height	Seat Dimension	Floor to Seat Height	Floor to Support Bar Height	Weight Capacity	Wheel Dimension	Net Weight	Carton Size
Measures in Cm											
01.06.550S	71	46	50.5	113/150	52x21	46	97.5	130	10 (4")	13	78x15x100
01.06.550A	71	46	50.5	100/134	52x21	46	97.5	130	10 (4")	13	78x15x100
01.06.550T	71	46	50.5	101/137	52x21	46	97.5	130	10 (4")	13	78x15x100

## ASSEMBLY

The frame of the rollator is divided into 7 parts:

SKU 01.06.550S

SKU 01.06.550A

**Base frame assembly:**

Insert 3 (at the top, with the fixing knobs facing outwards) and 4 (at the bottom, with the fixing knobs facing outwards) in the lateral supports 1 and 2.

**WARNING!**

Tighten all four fixing knobs firmly.

Lock the frame (1 + 2 + 3 + 4) by inserting the U-shaped support (piece no. 5) into the holes on the piece 3.


**Attach the supports:**

Insert 6-7 to the newly assembled frame.

**WARNING!**

Screw the two side handles firmly, this will also allow the height adjustment of the supports.

**The frame of the rollator is divided into 6 parts:**

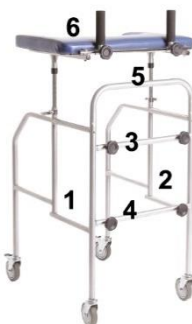SKU 01.06.550T

**Base frame assembly:**

Insert 3 (at the top, with the fixing knobs facing outwards) and 4 (at the bottom, with the fixing knobs facing outwards) in the lateral supports 1 and 2.

**WARNING!**

Tighten all four fixing knobs firmly.

Lock the frame (1 + 2 + 3 + 4) by inserting the U-shaped support (piece no. 5) into the holes on the piece 3.

**Attach the support:**

Insert 6 to the newly assembled frame.

**WARNING!**

Screw the two side handles firmly, this will also allow the height adjustment of the supports.

## USE

- Start pushing when you are in an upright position and both hands on the handle (see figure below).

- Make sure you have dry hands before using the walker to avoid the risk of slipping.
- Walk keeping your position upright with your eyes facing forward and not down.
- Do not use the walker to go over steps.

## CLEANING

Use a damp cloth with mild soap, the water temperature not above 30°C. Dry well before use. If necessary, disinfect the product using a common disinfectant detergent. Do not use water and/or steam jet washing machines. Must not use acid, alkaline or solvent products such as acetone.

## GENERAL DISPOSAL CONDITIONS

When disposing of the device, never use housing solid waste collection systems.

It is necessary to dispose of the device through the common ecological islands common for the recycling of used materials.

## WARRANTY

DEMARTA products are guaranteed from material or manufacturing faults for 2 years from the purchasing date.

DEMARTA does not guarantee for damages caused by the following condition: natural disaster, not authorized maintenance and failure to adhere to the guidelines and instructions for use.