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USER MANUAL
FOREARM WALKER
SKU 01.06.600

INTRODUCTION

Thank you for purchasing one of our products. The walker have been designed and manufactured to meet all your needs for practical, correct and safe use. This manual contains some suggestions for a correct use of the device and valuable advice for your safety. We recommend to read all the manual carefully before using the item. In case of any doubts, please contact the dealer, who will be able to advise you correctly.

Note: Check all parts to verify it won't be any shipping damage. In case of any damages, do not use it. Contact the dealer for further instructions.

DECLARATION OF CONFORMITY

DEMARTA VIRGINIO SAS declare under sole responsibility that the products that belong to walking aids are in accordance with the European Directive on Medical devices (93/42/EEC).

DEMARTA VIRGINIO SAS guarantees and declared under sole responsibility what follow:

- The devices under discussion satisfy the essential requisites requested by the I enclosure directive 93/42/CEE as laid down by the VII enclosure of the above mentioned Directive.
- The devices under discussion belong to class I.
- It is NOT PERMITTED to use this devices outside the intended use envisaged by DEMARTA VIRGINIO SAS.
- The articles also comply with all European standards applicable to them.

INTENDED USE

The walker are indicated in the walk rehabilitation for elderly or subjects with functional limitations of various kinds. They offer stability, ease of use and improve the sense of balance. This aid is designed for a single user, in indoor environments.

WARNINGS

- The walker must be used only by patients with a suitable motor ability that guarantees their use without risks.
- Consult your doctor or therapist to determine the suitability of the product, the right adjustment and the correct use of the walker.
- Your doctor or therapist should controll the height-adjustment of the walker, the correct activation of the brake, the correct assembly in general.
- Failure to follow these simple warnings may result in personal injury.
- The walker can only be used on a solid, flat surface.
- Do not use the walker to climb stairs.
- All the wheels must be in contact with the floor at any time during use to guarantee the correct balance.
- Do not hang anything on the walker frame to avoid distortion of it.

- Before using the device, always check:
 1. That all parts of the walker are securely locked.
 2. Check that all screws are tight.
 3. Check for proper brake operation.

TECHNICAL FEATURES

Code	Length	Inside Width	Width	Height	Seat Dimensions	Floor to Seat Height	Weight Capacity	Front Wheel	Rear Wheel	Net Weight	Carton Size
Measures in Cm											
01.06.600	77	45	64	105/122	52x21	59	120	10 (4")	10 (4")	12	77x20x97

DESCRIPTION

- Steel frame with height-adjustable padded forearm support and two ergonomic handles for a more comfortable grip.
- Four rotating wheels Ø 10 cm, rear wheels with brake.
- Padded seat included.
- Maximum Capacity: 120 Kg.

ASSEMBLY

The frame of the walker is divided into 4 parts:

nr 2 side parts, nr 1 horizontal tube, nr 1 brachial support.

To assemble, fix the two side parts to the horizontal tube, everything is fixed at the top by the 4-pin antibrachial support.

WARNING!

Tighten all the fixing handles firmly.

USE

- Start pushing when in an upright position, with the antibrachial supports adherent to the body, and both hands on the ergonomic handles.
- Make sure your hands are dry before using the rollator to avoid any risk of slipping.
- Walk while standing upright with your eyes facing forward and not down.
- Do not use the rollator to overcome steps.

CLEANING

Use a damp cloth with mild soap, the water temperature not above 30°C. Dry well before use. If necessary, disinfect the product using a common disinfectant detergent. Do not use water and/or steam jet washing machines. Must not use acid, alkaline or solvent products such as acetone.

GENERAL DISPOSAL CONDITIONS

When disposing of the device, never use housing solid waste collection systems.

It is necessary to dispose of the device through the common ecological islands common for the recycling of used materials.

WARRANTY

DEMARTA products are guaranteed from material or manufacturing faults for 2 years from the purchasing date.

DEMARTA does not guarantee for damages caused by the following condition: natural disaster, not authorized maintenance and failure to adhere to the guidelines and instructions for use.