## MANUALE D'USO STREAM AIR

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## USER MANUAL STREAM AIR

## INTRODUCTION

Thank you for choosing Demarta Virginio. The products have been designed and built to meet all your needs for a practical, correct and safe use. This manual contains some small suggestions for a correct use of the device of your choice and some precious advice for your safety. We recommend that you carefully read the entirety of this manual before using the item you purchased. If in doubt, please contact your dealer, who will be able to help and advise you correctly.

The manufacturer reserves the right to make changes to the device and to this manual without prior notice in order to improve its characteristics.

## GENERAL WARNINGS

- Use the product in the manner indicated in this manual. If doubts persist, contact DEMARTA VIRGINIO SAS.
- Always evaluate the compatibility of the product with the characteristics of the subject together with the doctor or therapist.
- Always check the integrity of the product before each use, even when new. Do not use if damaged or worn.
- DEMARTA VIRGINIO SAS declines any responsibility for damage to property or persons resulting from improper use, not agreed with a professional figure, not corresponding to what is indicated in the user manual.
- Any accidents, anomalies, breakages must be reported to DEMARTA VIRGINIO SAS in order to monitor the quality and criticalities of the product.

## DECLARATION OF CONFORMITY

DEMARTA VIRGINIO SAS declares under its sole responsibility that the product made and traded by DEMARTA VIRGINIO SAS comply with the applicable provisions of the 2017/745 Regulation on Medical Devices of 5 April 2017.

For this purpose, DEMARTA VIRGINIO SAS guarantees and declares under its sole responsibility what follows:

- The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
- The devices ARE NOT MEASURING INSTRUMENTS.
- The devices ARE NOT MADE FOR CLINICAL TESTS.
- The devices are packed in NON-STERILE BOX.
- The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation.

## INTENDED USE

The cushion air is intended for the prevention of pressure sores in subjects who must maintain a sitting position for prolonged periods of time.

Intended use: the products have been made for use both in a sanitary and private environment, according to the medical prescription.

Limitations of use: due to the type of material used and the medical prescription.

## WARNINGS

- Beware of sharp objects ex. scissors.
- Avoid exposure to the sun and flames.
- Protect the pillow from contact with cigarette butts.
- The maximum weight supported by the air bubble cushion is 140 kg.
- Do not use the cushion if it is damaged.
- Make sure there are no objects resting on the cushion before sitting.

## DESCRIPTION

The STREAM AIR is a micro-interchange air bubble cushion with differentiated sectors. It is made of Neoprene to ensure the maintenance of a constant pressure inside. It is available with 2 valves which allow patients to regulate different pressures according to their needs and 6 or 10 cm high. The external cover is composed of a contact surface in Jersey (breathable two way stretchable Polyester), the sides in Polyester 3D (honeycomb structure) and an anti-slip base. The cover has a carrying handle, and zip on three sides 2 double layer Velcro laces on the side that allow to tie the cushion to wheelchairs and chairs. Inflation pump included.

The models available are:

- Profile 6 cm, one valve, capacity 140 Kg.
- Profile 10 cm, one valve, capacity 160 Kg.
- Profile 6 cm, two valves, capacity 140 Kg.
- Profile 10 cm, two valves, capacity 160 Kg.

## TECHNICAL FEATURES

Profile 6 cm • one valve		Profile 10 cm • one valve	
Code	Dimensions (cm)	Code	Dimensions (cm)
05.01.1CR88-6	40 x 40 x 6	05.01.1CR88-10	40 x 40 x 10
05.01.1CR98-6	46 x 40 x 6	05.01.1CR98-10	46 x 40 x 10
05.01.1CR99-6	46 x 46 x 6	05.01.1CR99-10	46 x 46 x 10
05.01.1CR48-6	50 x 46 x 6	05.01.1CR48-10	50 x 46 x 10


Profile 6 cm • two valves		Profile 10 cm • two valves	
Code	Dimensions (cm)	Code	Dimensions (cm)
05.01.2CR88-6	40 x 40 x 6	05.01.2CR88-10	40 x 40 x 10
05.01.2CR98-6	46 x 40 x 6	05.01.2CR98-10	46 x 40 x 10
05.01.2CR99-6	46 x 46 x 6	05.01.2CR99-10	46 x 46 x 10
05.01.2CR48-6	50 x 46 x 6	05.01.2CR48-10	50 x 46 x 10

## USE

- Place the cushion on the chair.
- Turn the cushion valve counterclockwise to open it.
- Connect the pump tube to the valve.
- Inflate the cushion until the cells are completely full.
- Turn the cushion valve clockwise to close it and then remove the pump tube.
- Sit on the cushion.
- Place your hands between your buttocks and the cushion.
- Open the valve to slowly release the air from the cushion.
- Close the valve after reaching a distance of three fingers between the buttocks and the bottom of the pillow.
- The cushion is now ready for use.

IMPORTANT! If there are 2 sections, use the same method described above to adjust the other as well.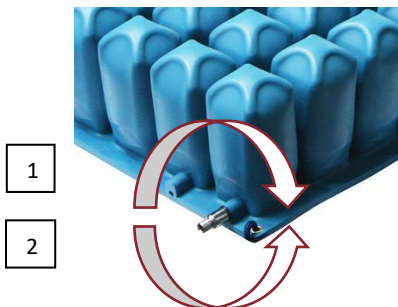

1-CLOSURE CLOCKWISE.

2-OPEN COUNTERCLOCKWISE.

**IMPORTANT! THERE ARE NO VALVE CLOSING CAPS.**

## COVER

Each air cushion is supplied with a cover to help protect the surface. The bottom of the cover is made of non-slip material that fixes the cushion on the chair and helps increase user stability.

To cover the cushion with the cover, follow the instructions below:

- Stretch the cover with the anti-slip base towards the chair / wheelchair seat.
- Open the zip and insert the cushion, bubbles facing upwards.
- Make sure the valves stay outwards (pass the valves through the respective holes on the cover).
- Close the zip carefully. The cushion is ready to use.

## CLEANING

Cushion Maintenance and Cleaning	
Disinfection possible with 5% diluted disinfectants, also alcohol based	
Cover Maintenance and Cleaning	
	Water temperature not above 40° C or 105° F
	Do not iron
	Do not bleach
	Do not use the dryer

## GENERAL DISPOSAL CONDITIONS

When disposing of the device, never use housing solid waste collection systems.

It is necessary to dispose of the device through the common ecological islands common for the recycling of used materials.

## WARRANTY

DEMARTA products are guaranteed from material or manufacturing faults for 2 years from the purchasing date.

DEMARTA does not guarantee for damages caused by the following condition: natural disaster, not authorized maintenance and failure to adhere to the guidelines and instructions for use.