GI-emme s.r.l.

- TKS2012-A
- TKS2012-B

SISTEMA ANTIDECUBITO

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1. GENERAL ADVISE OF SECURITY

CAUTION!:

PLEASE, SEE THE INSTRUCTIONS PUT ON THE PUMP FOR MATTRESSES

Instruction to be read before installing and using the antidecubitus system.

- On receiving your product please control that you have all the necessary parts and that there is nothing damaged due to transport or whatsoever. In case of damage contact the salesman.
- ◆ Please conserve all packaging for eventual transport in future.
- ◆ Do not interfere with system of the pump when in use and remove electric plug before moving it.
- ◆ Do not place antidecubitus mattresses near to heat.
- ◆ Do not use the pump neat water or wet surface and do not ever make wet.
- ◆ Do not place the pump on an instable surface.
- ◆ The right electrical power must be used, indicated on label which can be found on it.
- ◆ If there is any problem with the function of product, please contact qualified people concerned.

2. PUMP MACHINE FEATURES

The correct and safe functioning of the pump is assured only if in accordance with the following indications:

- ◆ Common machine, water proof.
- ◆ The a.m. machine is not to be used in presence of an anesthetic of inflammable substance in contact with the air or oxygen or idrogen manoxide.
- ◆ The pump is made for a constant functioning.
- ◆ A declaration that states that there are no electromagnetic interference in presence of other machines.

3. TECHNICAL DATA

PLEASE, SEE THE LABEL PUT ON THE PUMP

4. USE AND FUNCTIONING

PLEASE, SEE THE INSTRUCTION PRINTED ON THE PUMP

- ◆ Place the pump at the bed's feet on the floor under the bed or on to a stable surface, or hang it on the bed by using the special hanging on the bottom of the pump;
- ◆ Connect the alim, wire to a 220 Volt;
- ◆ Place the PVC mat (bubble or int) on to any traditional mat, taking care that on the floor are no parts or anything else that can damage the mat. The air connections must face the bottom of the bed;
- ◆ Connect both the tubes of the pump to the ones on the mat;
- ◆ Push the green button situated on the front of the pump;
- ◆ In case you are using a pump accessories with the pressure regulator, regulate it at the max capacity;
- ◆ Wait about 20/25 minutes so as to let the circuit reach the pressure(with bubble mat);
- ◆ Wait about 25/30 minutes so as to let the circuit reach the pressure (with interchange bars mat);
- ◆ Lay the patient on the pressured mat;
- ◆ In case of use of pump with pressure regulator please regulate it, as the following indications.

PATIENT WEIGHT PRESSURE SELECTOR		
PUMP MOD GM TKS 2012		
Kg.60	Kg.80	
	MIN	MAX

If use a bubble mattress type GM TKS A or similar the weight max is 100 Kg.

Once it is working the pumps cyclic and alternatively the separated element of the PVC mat. In this way, the pump produces a continuous and delicate message.

5. PUMP'S STOCKING AND DELIVERING

In order to have a safe stocking and storage of the pump machines, the original packaging had to be intact.

Extreme attention has to be paid also to the delivering of the pump machines since they are composed of parts that are assembled accuracy and care.

The ideal environmental conditions for the stocking at delivery of the a.m. are the following.

- ◆ Room temperature in between 10-40 °C
- ◆ Humidity in between 30-75%
- ◆ Atmospheric pressure in between 900-1060 hPa.

6. MAINTENANCE

The pump line had been studied to reduce as most as possible the necessary maintenance and to ensure a correct function in future without problems.

IMPORTANT: Before every interference on the appliance, disconnect the plug from main outlet.

- ◆ Wipe always the air pump with a damp cloth. Don't spill any liquid.
- ◆ The mattress can be cleaned with fresh water and most disinfectants. Don't use abrasive clothes and pointed or sharpened.
- ◆ Periodically clean the air pressure supply lined with a soft small stick. Make sure the nozzles are not blocked by dust.

CONDITIONS OF GUARANTEE:

Each component of the decubitus prevention system is guaranteed for manufacturing defects,

Important: keep always the certificate of guarantee together with proof of purchase.

The period of guarantee starts from the date of purchase. The guarantee includes replacement of defective parts and the labour necessary for their replacement. The guarantee is valid only when the appliance had been used according to the manufacturer's instructions and has not been tampered with.

The guarantee is NOT valid:

- ◆ When the appliance works in an irregular way;
- ◆ In case of running defects due to an incorrect installation of the system;
- ◆ In case of damages caused either by a wrong use, or an incorrect management of the system or repairs done by not authorized personnel;
- ◆ In case of damages caused by normal wear and tear.

In case of failure:

- ◆ Deliver the appliance for repairing to the dealer or authorized personnel, packing the appliance carefully in its original box;
- ◆ Enclosed the certificate of guarantee, together with proof of purchase and a request of repair service describing in sufficient details, the nature of the defect.

The guarantee is valid for a period of 12 months, except special cases fixed apart.

PROBLEMS AND SOLUTIONS:

The inflating period exceeds the estimated times	Wait 10/15 minutes more to see if the appliance works correctly; if at this point the mattress was not fully inflated, it would be better to contact the dealer.
The air pump doesn't light up	ONLY WITH EXTERN FUSE Verify the conditions of the fuse and, if necessary, replace it. If the defect continues, please contact the dealer
Noise: during the night noise's level diminishes, so it can be heard only a light buzzing due to air pump	Advices to reduce the noise: 1. Always place the air junctions of mattresses turned to the foot of the bed. 2. Place the air pump on the floor underneath the bed or on a steady surface

We declare that the air pumps of the decubitus prevention system belong to the I class, type BF and care manufactured in compliance with the requirements set in the Council Directive 93/42/EC.

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GI-emme S.r.l.