MANUALE D'USO
EASY SOFT
CODICE 02.02.672

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USER MANUAL
EASY SOFT
SKU 02.02.672

INTRODUCTION

Thank you for purchasing one of our products. The raised toilet seats have been designed and manufactured to meet all your needs for practical, correct and safe use. This manual contains some suggestions for a correct use of the device and valuable advice for your safety. We recommend to read all the manual carefully before using the item. In case of any doubts, please contact the dealer, who will be able to advise you correctly.

Note: Check all parts to verify it won't be any shipping damage. In case of any damages, do not use it. Contact the dealer for further instructions.

DECLARATION OF CONFORMITY

DEMARTA VIRGINIO SAS declare under sole responsibility that the products are in accordance with the European Directive on Medical devices (93/42/EEC).

DEMARTA VIRGINIO SAS guarantees and declared under sole responsibility what follow:

- The devices under discussion satisfy the essential requisites requested by the I enclosure directive 93/42/CEE as laid down by the VII enclosure of the above mentioned Directive.
- The devices under discussion belong to class I.
- It is NOT PERMITTED to use this devices outside the intended use envisaged by DEMARTA VIRGINIO SAS.
- The articles also comply with all European standards applicable to them.

INTENDED USE

Bath safety are all those devices that can be used by a disabled or elderly person to prevent, compensate or eliminate the difficulty in performing all the functions in the bathroom.

WARNINGS

- Keep the cardboard packed away from any heat source.
- The life of the device is determined by the wear of parts that cannot be repaired and / or replaced.
- Always pay attention to the presence of children.
- Use with only one patient is recommended.
- Sanitize the device before each use.

TECHNICAL DATA

Code	Height (cm)	Capacity (Kg)
02.02.672-2	5	130
02.02.672-4	10	130

DESCRIPTION

The raised toilet seat EASY SOFT is composed of soft antioxidant and water repellent PU. It is fixed thanks to a double lip that adapts perfectly to the thickness of any toilet. The anatomical shape and the soft consistency make it particularly comfortable.

INSTALLATION

1. Lift the toilet lid and the tablet.
2. Position EASY SOFT perpendicularly to the toilet and push firmly until it stops.
3. Carefully check the stability before use it.
4. If necessary, request supervision or help during use.

The EASY SOFT is positioned on the ceramic of the toilet and does NOT require any fixing screws or locking systems.

PRECAUTIONS

- Do not exceed the maximum capacity.
- Before use, always make sure the toilet seat is well fixed to the toilet and that there is no damage that could compromise the stability of the product.
- Caution in case of wet or slippery floor.

CLEANING

Washing can take place with warm water and neutral soap
Disinfection can take place with simple products of normal domestic use

GENERAL DISPOSAL CONDITIONS

When disposing of the device, never use housing solid waste collection systems. It is necessary to dispose of the device through the common ecological islands common for the recycling of used materials.

WARRANTY

DEMARTA products are guaranteed from material or manufacturing faults for 2 years from the purchasing date.

DEMARTA does not guarantee for damages caused by the following condition: natural disaster, not authorized maintenance and failure to adhere to the guidelines and instructions for use.