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## INDEX

1. Introduction
3. Shipping
  - Assembling
  - Use specifications
  - Safety information
4. Components
5. Check before use
  - High chair use
  - Brake - Push brake
6. Depth seat adjusting
7. Transport by car
8. Wheelchair maintenance
9. Repairs
10. Measures
11. Optionals
13. General safety rules
14. Portable weight
  - Transfer from the wheelchair
15. Crossing narrow passages
  - Using elevators and lift bridges
  - Transportation by car or other means of transport
16. Declaration of compliance
17. Warranty
18. Plate and EC label
19. Maintenance log
22. Data sheet
23. Section for retailers

## Instructions for use and maintenance

### Introduction

The below introduced product, BASICA, is the result of twenty years of experience that has seen our company engaged in a process of continuous research to improve and refine techniques for the design and manufacture of our products; everything is possible using Technical Orthopedics and advice from experts in the field of rehabilitation, education and support.

This manual provides instructions for the use of this model. Please read the following instructions before using the aid for the first time.

The special care taken in the design and manufacture of this product ensure you a high degree of safety and comfort.

Use your wheelchair by following the instructions contained in this manual.

**Model BASICA:** External base. Structure easily attachable to the seating system. Foldable or reducible for easy transportation.

Ability to accommodate seating systems appropriate to the evolving user situation. Regulating the tilt with the device that allows easy tilting of the system even with the person sitting.

**Main clinical indications:** indicated for patients with severe disability, secondary to motor impairments that result in a major failure posture.

**Base equipment:** available in different sizes, equipped with tilting system from 0 ° to 32 ° reclining from 0 ° to 20 ° with adjustable angles, adjustable backrest height, adjustable seat depth and width.

#### **Optionals:**

customization options through the choice of accessories to meet your needs of comfort and safety: range of headrest with a new control system, multifunctional footboards adjustable in various positions with reinforced fixing system, adjustable tension backrest or aluminum, hemiplegic armrest, tables of various characteristics, lateral push padded, retractor, footboard, pelvic girdle 45 ° and push bar for the passenger height-adjustable, height adjustable armrests and rails and shaped cushions seat and back.

## Introduction

### Symbols

Often, while reading the manual, you may meet these symbols of particular importance or interest.

Indicates to pay special attention hazard in general

key nuts or bolts from 6/19 mm.

Allen key for hexagon socket screws 6

Basic features of:

- tilting seat system
- reclining backrest
- drum brakes

by this way allowing more confort.

Moreover: tipper backrest, removable armrest, reduce the involved space to the minimum, in order to simplify the transport.

## Shipping

All Nuova Blandino products are tested in-house to verify compliance with all specifications and then are shipped in extra cardboard boxes.

Please, check the device soon after receiving, in order to verify if any damage occurred during transport, and if possible, in the presence of the carrier.

If the purchased product was damaged during the transport, the damage must be reported immediately, with an official written claim to the delivery note.

All damaged parts (if removable) or the whole wheelchair must be delivered carriage free to Nuova Blandino Srl, within 8 days from the date of receipt, together with a copy of the receipt.

No damage will be acknowledged unless the aforementioned procedure is followed, or if the damage is reported after the receipt note is issued.

## Assembling

The orthopedic specialist will deliver your multifunctional highchair ready for use, adjusted according to your individual necessity.

## Use specifications


The aid is designed to be used indoors and outdoors. It is intended exclusively to transport a person in the seat, and

not as a means of traction, transport, or otherwise.

## Safety information

All information for a safe use of external base are shown in the second section of this handbook.

## Components

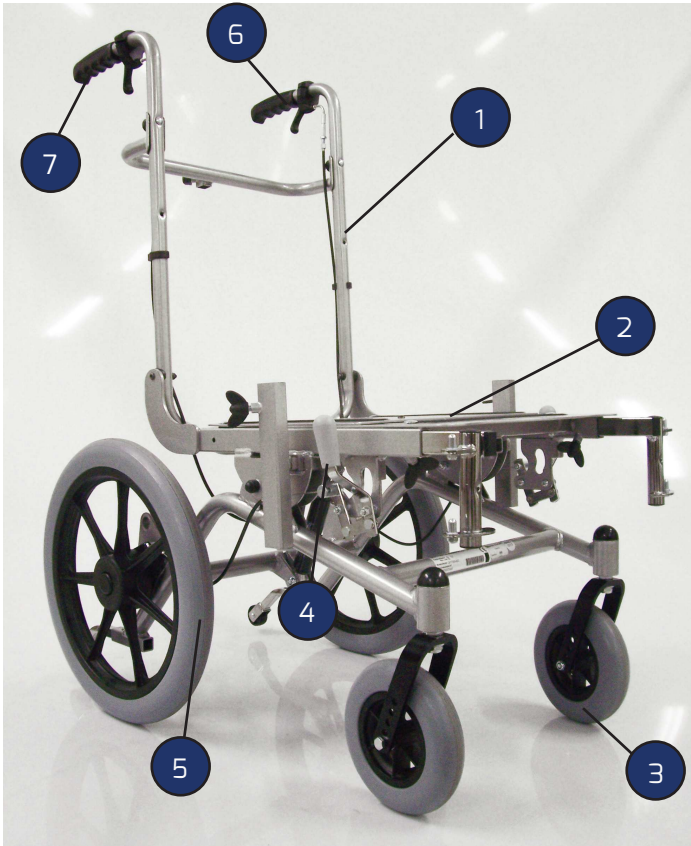1. Reclining backrest
2. Tilting adjustable seat
3. Front wheel
4. Drum brake lever
5. Rear wheel Ø 400
6. Tilting seat lever
7. Reclining backrest lever

## Check before use

Before you can use the aid, check that all parts are properly assembled and tightened.

Check the correct operation of the brake pressure. By placing the brakes on both wheels, and imparting a slight force to the chair forward and backward: the wheels should not rotate.

## Using the wheelchair

The aid is ideal for indoor leisure pool, as long as the ground support is solid and not slippery.

Use the aid initially with great prudence. Read carefully the second part of the manual "Safety Information".

## Brake - Drum brake -

Advantages: braking action regardless of the profile of the tire.

Braking is independent of the pressure of the tire.

Braking action even with wet tire.

## Depth seat adjusting

- loosen the screws indicated in the figure
- move forward or backward the seat plate and find the correct position
- tighten the screws


loosen the screws  
and slide the plates

## Transport by car

Carrying the wheelchair in the car:

- remove the pin with ring (fixed piston) (a)
- break down the backrest
- remove the anti-tip wheels.

To reassemble the wheelchair just follow all the steps in reverse order.

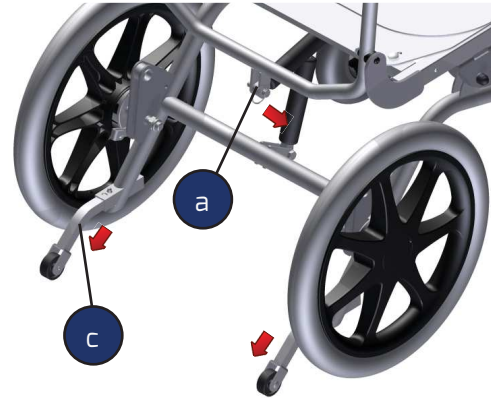## Wheelchair maintenance

### Yearly checks

Maintenance should be performed once a year. It is highly recommended to check the state of the bearings, since sand, dust and water may damage them. Lightly lubricate with bicycle resin-free oil.

The frame may be cleaned with hot water and soap (or household detergent) only. Plastic parts should be cleaned with detergents specific for this material.

**Caution!**

Do not use high pressure cleaning equipment!

The aids should not be cleaned with harsh detergents.

**Caution!**

All components which come in direct contact with the skin must be replaced if the wheelchair is given to another user.

## Repairs

### **Repairs**

To repair, trust your Nuova Blandino Srl dealer, who was educated in our factory according to our specifications, and is able to offer advice, support and repair services.

### **Spare parts**

To be ordered through your dealer. For the purposes of proper delivery of replacement parts is necessary to indicate the number of the frame relative to the wheelchair. For safety reasons, use original parts and accessories Nuova Blandino S.r.l.

### **Disposal**

Disposal should be in accordance with the relevant rules of national law.

## Measures


cm 36 40 45 50


cm 49 53 58 63

cm 74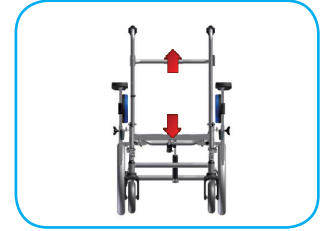

cm from 54 to 60

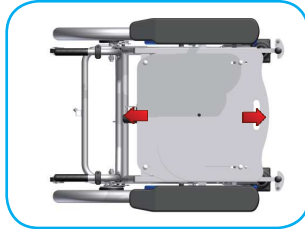cm from 42 to 55

cm 42


Kg 21


Max Load. Kg 120

## Optionals

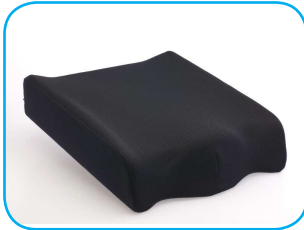Shaped seat cushions

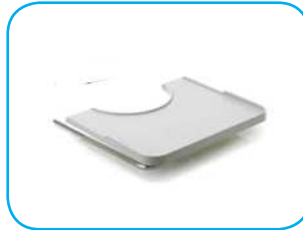Anatomic tray table with fixing brackets

Rectangular table with fixing brackets

Padded anatomical table with fixing brackets

Shaped backrest

Tension adjustable backrest

Multifunctional legrest

Footboard

Strap legrest cm 15

Strap legrest cm 25

Abductor pad

Lateral support pad

## Optionals

Simple belt

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Cintura pelvica grande

Armrest for hemiplegic

Retractor band

Safety belt

Y shaped belt

multifunctional headrest

adjustable armrests

## General safety rules

Use the wheelchair only according to the instructions.

**Warning: the model described in this manual can not be used as well as it is purchased, but should be complemented with adequate seating systems prescribed by orthopedic technicians.**

To station, get on and off the wheelchair, make sure the ground is flat and stable and secure the wheelchair to prevent it from moving by activating both brakes.

If exposed to direct sunlight, the covering and the upholstery of the seats, the armrests, the legrests and the handles may reach temperatures over 41° C. Direct contact with unprotected skin may cause injury. In order to avoid the over-heating of said parts, exposing the parked wheelchair to direct sunlight is not recommended.

The dealer will be happy to adjust your BASICA to suit your individual needs, by maximum respect for your safety.

When carrying objects, make sure that these do not affect or restrict the handling of highchair.

Check the pressure and the wear of the tires and check the brakes.

## Portable weight

The maximum allowable load is kg. 120 and can be found in the instructions for use and label of the aid

**The aid can carry only one person.**

## Transfer from wheelchair

Getting off the wheelchair may be dangerous, if no assistance is available.

It requires:

- considerable strength
- good coordination abilities
- a safe place
- much training

The knowledge of all of the possible dangers.

Let yourself be helped by a man. Discuss with him the exercise and the actions that may lead to unexpected dangers, for example. any stumbling points.

## Crossing the narrow passages

When crossing narrow passages, such as the entrance of a building, a door, etc., pay attention to your elbows.  
**Danger of injury!**

## Using elevators and lift bridges

Lifts and elevators pose a particular danger to the user.

**Warning!**

Pay attention to the size of the lift and pack of the aid to avoid getting stuck.

## Transportation by car or other means of transport

The wheelchair should never be used as a seat when you use a car or other means of transport. Provide the disassembly of the wheelchair (as specified in the instructions for use) and store it in the car, or at least secure it in ways to avoid accidents in case of dangerous maneuvers.

Take a proper seat inside the car or the means of transport and fasten your seatbelt.

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Responsible for designing, manufacturing, packaging, and labeling of this product, that falls within the category of

EXTERNAL BASE

**DOES GUARANTEE AND DECLARE**

That the above aid, and entered into service in the market as from the date of this certification, fully complies with the requirements set out in Annexes I and VII of the EC Directive 2007/47 EEC and DL 37 of January 25, 2010, Italian transposition of the said Directive.

The products listed above are classified NOT Invasive Medical Devices Class I in accordance with Annex IX and X of Directive 2007/47 EEC.

Concerning the remaining guidelines and standards applicable to the product , the complete “EC” declaration of conformity is available on the website [www.nuovablandino.com](http://www.nuovablandino.com)

*The afore-mentioned products:*

- *are not measuring instruments*
- *are marketed in non-sterile packaging*

This is to certify that the above products were manufactured in compliance with the technical provisions set by the company and that all checks and trials were carried out before commercialization.

Settimo, 26th November 2015

## WARRANTY

On our external base, we provide a warranty period of two years from the correct completion of the certificate of purchase (see page 23).

The warranty covers any manufacturing flaw or malfunction on guaranteed products.

Damages caused by carelessness or improper use are excluded from the warranty. Authorized resellers will provide additional information and quotations if any repair is needed. For safety reasons, only spare parts or accessories authorized by Nuova Blandino S.r.l. should be used. The use of unauthorized accessories is a reason of immediate decline of warranty.

In order to guarantee it works properly, the highchair must be kept clean, in order and maintained as described in the handbook of use.


Caution:

Nuova Blandino does not assume any responsibility for damage caused by the combined use of our aids with parts of all kinds from other manufacturers (eg. Trailer units, electrical devices), which may involve significant dangers - except that a given product has not been expressly authorized in writing by the New Blandino S.r.l.

*Nuova Blandino S.r.l. will not be liable for any damage to persons, things, animals, and any third party due to modification or misuse of the wheelchair.*

## Plate and EC label

The EC label (see picture) is printed on the highchair on the front side of the crossbar under the seat.

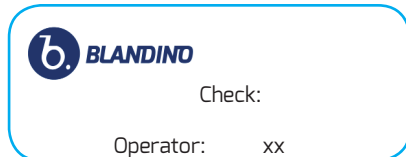## Maintenance log

Repair date	Repair date
Replaced or repaired parts	Replaced or repaired parts
Stamp or signature of the operator	Stamp or signature of the operator
Stamp or signature of the inspector	Stamp or signature of the inspector

## Maintenance log

Repair date	Repair date
Replaced or repaired parts	Replaced or repaired parts
Stamp or signature of the operator	Stamp or signature of the operator
Stamp or signature of the inspector	Stamp or signature of the inspector

# Maintenance log

Repair date	Repair date
Replaced or repaired parts	Replaced or repaired parts
Stamp or signature of the operator	Stamp or signature of the operator
Stamp or signature of the inspector	Stamp or signature of the inspector

## Advisory notices

The device described in this manual is EC labeled and complies with the regulations laid down in EEC Directive 2007/47 and L.D. 37 of 25 January 2010, for medical devices. It is classified as a Class 1 non-invasive medical device, in compliance with annex IX and X of the above-mentioned Directive.

- **Intended use**

The device is supplied ready for use by technical orthopedics and adjusted according to the user's needs.

The external base is designed for both indoor and outdoor use. It is intended to accommodate one single person on the seat when the legrests and the lateral pads are mounted, and should not be used as a means of traction, transport or otherwise.

Carefully read the instruction manual for a correct use of the device.

- **Check before use**

Before using this BASICA, make sure that all parts are properly assembled and tightened, in particular the footrests. For safety reasons, if visible flaws are detected on the product, all damaged parts must be replaced.

- **How to clean**

*Do not use high pressure cleaning devices! Mechanical damage may be caused by water jets.*

*Chairs must not be cleaned with aggressive cleaning products.*

- **Lifespan of the device**

This external base may be used for a variable period of time, depending on the time and the frequency of use. A correct use and a proper maintenance of the wheelchair may extend its lifespan; on average, it is expected to be fully functional for 5-7 years.

- **Annex VII-Declaration of compliance-**

Nuova Blandino S.r.l. performs and updates a systematic post-production evaluation of the devices, on the basis of the provisions referred to in Annex X. It also provides support for the implementation of any corrective measure that may be necessary to carry out, bearing in mind the nature and the risks that may affect the product.

Malfunctions or any lack of information which has resulted in accidents or other type of damage on the user must be promptly communicated to Nuova Blandino S.r.l. by the user or anyone who acts in his or her stead.

FOR TECHNICAL OR USE INFORMATION, OUR EXPERT IS AVAILABLE MONDAY TO FRIDAY, FROM 08:30 TO 12:00 HOURS AND FROM 13:30 TO 7:00 HOURS. CALL +390112625140.

**RETAILER STAMP**

--

**DATE OF PURCHASE**

--	--	--

**REGISTRATION DATE**

--	--	--

N.B.: THIS MANUAL MUST BE PRODUCED WHENEVER ASSISTANCE IS NEEDED

Nuova Blandino S.r.l.  
Strada Cebrosa, 24 - 10036 Settimo Torinese  
Torino - Italy  
Tel. +39 011 2625140 - fax +39 011 2625141  
[www.nuovablandino.com](http://www.nuovablandino.com) - [inforb@nuovablandino.com](mailto:inforb@nuovablandino.com)

Commitment and technology for people with special needs